Instructions for Use

Product description

Zfx ScanDry lab is used to achieve optimum recordings for CAD/CAM restorations (e.g. inLab®). Zfx ScanDry lab balances out the different optical properties of the stone model and thus permits optimum recording using a scanner. With the atomizing nozzle, an optimum covering layer with a detailed representation of the edges is achieved easily and efficiently with only a short spray discharge.

Indication

Extra-oral application (plaster die / plaster model) for lab-side use

Application restrictions

Application of the spray on unsuitable and dried surfaces

Contraindication

intra-oral application

General note

Conduct recording using the scanner according to the manufacturer's Instructions for Use. Failure to observe the application restrictions or stipulated working steps may result in failure.

Composition

Pigment suspension in ethanol. The propellant is a mixture of propane/butane.

Side effects

Side effects are not known to date. In individual cases, allergic reactions may occur.

Processing

Preparation for spraying

1. Attach spray cannula to the can and turn the nozzle to the correct position.
2. The nozzle can be rotated 360° to achieve an optimum position to reach the areas to be spray coated.
3. The spray cannula can be slightly bent. However, never kink it at a right angle.
4. Clean the surface to be spray coated and dry thoroughly.
5. Shake well before use.

Spray procedure

1. Carry out one short spray discharge to achieve a thin, complete wetting.
2. The can must always be kept upright during spraying.
3. The spray head must be entirely pressed down during spraying to prevent premature clogging of the atomizing nozzle.
4. The distance from the nozzle to the surface to be coated should be approximately 4 cm.

Cleaning of the surfaces

1. Once the recording has been completed, the surface can be cleaned using water and steam jet. To achieve thorough cleaning of the surfaces, the procedure may have to be repeated.
2. Blow out the atomizing nozzle using water and steam jet to prevent it from clogging.

Caution

The container is under pressure. Protect from direct sunlight and temperature over 50°C / 122°F. Do not open by force or burn it, even once the container is empty. Keep it out of the reach of children.

Date information prepared

06/2012

Made in Germany

Zfx GmbH
Kopernikusstraße 15, 85221 Dachau, Germany
Tel. +49 81 31 33 244 - 0, office@zfx-dental.com
www.zfx-dental.com

This material has been developed solely for use in dentistry. Processing should be done strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions of the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the instructions. Description

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